

Note: If a battery is already in the device, press and hold the **Event** button for 10 seconds to turn off the device before removing the battery using the pull ribbon within the battery compartment. **Replace only with a fully charged battery.**



Step 1: Remove the battery cover by pressing down on the lines and pushing away from the device.



Step 2: Ensure the blue arrow on the battery and the blue arrow in the compartment match.



Step 3: With the battery label facing up, set the battery into the compartment at an angle on top of the pull ribbon. **DO NOT** force the battery down flat on top of the battery contacts.



Note: The battery will sit at an angle in the battery compartment.



Step 4: Firmly press the other end of the battery down into the compartment. The battery will now sit flat in the battery compartment on top of the pull ribbon and the device will power on.



Step 5: Replace the back cover by inserting it at an angle to align the tab with the device. Once aligned, press down and push close to close the battery cover.

When finished using the TeleSense device, return both the device and this document to your clinician.